

The impact of the 80-gene signature on pCR and chemotherapy treatment decisions in early-stage breast cancer: A FLEX analysis

A.M. Brufsky¹, W. Audeh², H. Ramaswamy², J. Klinkhamer³, S. Meijssen⁴, N. Stivers², A. Menicucci², J. O'Shaughnessy⁵

¹Hematology-oncology, UPMC Hillman Cancer Center, Pittsburgh/United States of America, ²Medical Affairs Department, Agendia Inc., Irvine, CA, USA, ³Market Access, Agendia NV, Amsterdam, Netherlands, ⁴Medical Affairs, Agendia NV, Amsterdam, Netherlands, ⁵Baylor University Medical Center, Texas Oncology, Sarah Cannon Research Institute, Dallas, TX, USA

Introduction

- Hormone receptor-positive (HR+), HER2-negative early-stage breast cancer (EBC) displays notable heterogeneity and diverse responses to neoadjuvant chemotherapy (NCT)
- Genomic profiling has demonstrated utility in guiding pre-operative treatment decisions by predicting the probability of achieving a pathological complete response (pCR) and chemosensitivity¹
- Breast cancer subtyping based solely on receptor status by IHC may not fully capture tumor biology or sufficiently guide personalized treatment decisions^{1,2}
- The Blueprint[®] 80-gene signature provides insight into the intrinsic subtypes of EBC, identifying patients that have tumors that may be more likely to benefit from more aggressive treatments
- This study evaluated pCR rates to NCT by Blueprint molecular subtypes in MammaPrint[®] High Risk tumors and the role of Blueprint in guiding adjuvant chemotherapy (CT) treatment decisions from patients enrolled in FLEX- a prospective, longitudinal, observational real world data study

Methods

Study Cohort

- Patients included in this analysis were enrolled in the FLEX trial (NCT03053193), received MammaPrint and Blueprint, and had HR+HER2- EBC
- The analysis included patients eligible for CT with MammaPrint High Risk tumors, which were grouped into patients who received:
 - NCT and had available pCR data (N=401)
 - Adjuvant CT with treatment recommendation data based on Blueprint results (N=868)
- MammaPrint High Risk of recurrence results were further characterized by Blueprint molecular subtyping signature as Luminal B or Basal^{1,2}

Statistics

- Differences in clinical characteristics, pCR rates, and treatment differences were evaluated by Chi-Squared test or Fisher's exact tests
- P-values of less than 0.05 were considered significant

Results

Table 1. Clinical Characteristics of FLEX patients with HR+HER2- disease treated with NCT and available pCR data


	Luminal B (N=312)	Basal (N=89)	Overall (N=401)	P-value
Age (Years)				
Mean (SD)	54 (± 12)	54 (± 13)	54 (± 12)	0.971
Menopausal Status				
Post-	182 (61.9%)	54 (65.1%)	236 (62.6%)	0.877
Pre-/Peri-	112 (38.1%)	29 (34.9%)	141 (37.4%)	
Race				
White	223 (75.9%)	61 (76.3%)	284 (75.9%)	0.806
Black	43 (14.6%)	15 (18.8%)	58 (15.5%)	
Latin American	19 (6.5%)	1 (1.3%)	20 (5.3%)	
AAPI	6 (2.0%)	3 (3.8%)	9 (2.4%)	
AIAN	1 (0.3%)	0 (0%)	1 (0.3%)	
Mixed	2 (0.7%)	0 (0%)	2 (0.5%)	
Tumor Stage				
T1	55 (22.3%)	21 (29.6%)	76 (23.9%)	0.887
T2	137 (55.5%)	39 (54.9%)	176 (55.3%)	
T3	39 (15.8%)	8 (11.3%)	47 (14.8%)	
T4	16 (6.5%)	3 (4.2%)	19 (6.0%)	
Lymph Node Status				
LN-	82 (33.9%)	43 (65.2%)	125 (40.6%)	<0.001
LN+	160 (66.1%)	23 (34.8%)	183 (59.4%)	
Grade				
G1	19 (6.5%)	0 (0%)	19 (5.1%)	<0.001
G2	147 (50.5%)	7 (8.3%)	154 (41.1%)	
G3	125 (43.0%)	77 (91.7%)	202 (53.9%)	

Data presented as n (%); Unknown values excluded; N, sample size; AAPI, Asian American and Pacific Islander; AIAN, American Indian or Alaska Native

CT treatment Recommendation Cohort (N=868)

- 93% of the patients had tumors classified as Blueprint Luminal B and 7% as Basal (**Table 2**)
 - Among Basal, 88.1% were LN-, 41.5% were ER >50%, and 81.7% were grade 3, compared to 79.6% (p = 0.283), 97.2% (p<0.001) and 18.8% (p < 0.001) for Luminal B, respectively
 - Only 13.2% of Basal tumors had an ER expression of <10%
 - 98.4% of patients with Basal tumors received CT vs. 76.9% of Luminal B (p=0.19)
 - Basal-type patients are more likely to receive neoadjuvant therapy (27.4% vs. 11.9%; p=0.006)
- CT recommendations showed that patients with Basal tumors were twice as likely to receive AC-T (50.0% vs. 23.9%; p=0.001) and less likely to receive TC (37.5% vs. 65.2%; p=0.13) (**Figure 2**)
- The Basal subtype was more frequently treated with CT (98.4%) and neoadjuvant therapy (27.4%) compared to Luminal B (77%, p<0.001; 11.9%, p=0.006, respectively) and with a more aggressive CT regimen than docetaxel/cyclophosphamide (Basal: 50.0%, Luminal B: 25.7%; p<0.001)
- When restricted to Grade 3 tumors, Basal-type tumors were more often treated with AC-T (39.6% vs. 21.2%; p=0.02) compared to Luminal B (data not shown)
 - Similarly, among tumors with ER staining >50%, Basal were more likely to receive AC-T (40.9% vs. 16.7%; p=0.04) compared to Luminal B

Figure 1. pCR data Luminal B vs Basal tumors

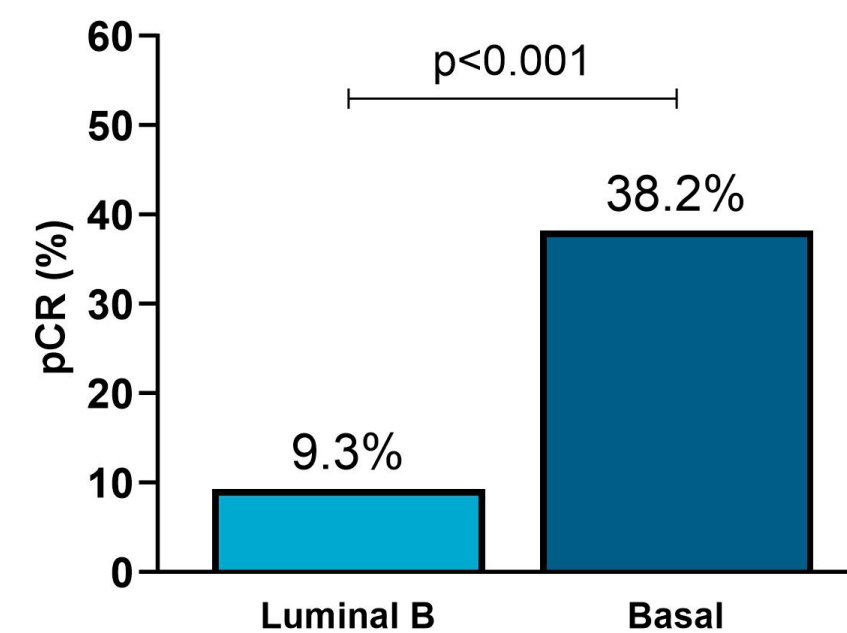

NCT-treated cohort with pCR data (N=401)


- Blueprint classified 78% of tumors as Luminal B and 22% as Basal (**Table 1**)
 - LN+ was more common in Luminal B (66.1%) vs Basal tumors (34.8%; p<0.001)
 - More Basal tumors were grade 3 (91.7%) compared to Luminal B (43.0%; p<0.001)
- Patients with Basal tumors were significantly more likely to achieve a pCR (38.2%) vs Luminal B (9.3%; p<0.001) (**Figure 2**)

Table 2. Clinical Characteristics of FLEX patients with HR+HER2- disease in the treatment recommendation cohort

	Luminal B (N=806)	Basal (N=62)	Overall (N=868)	P-value
Age (Years)				
Mean (SD)	61 (± 11)	57 (± 12)	61 (± 11)	0.078
Race				
White	624 (82.4%)	38 (65.5%)	662 (81.2%)	0.114
Black	81 (10.7%)	15 (25.9%)	96 (11.8%)	
Latin American	34 (4.5%)	3 (5.2%)	37 (4.5%)	
AAPI	16 (2.1%)	2 (3.4%)	18 (2.2%)	
AIAN	2 (0.3%)	0 (0%)	2 (0.2%)	
Menopausal Status				
Post-	615 (81.3%)	45 (80.4%)	660 (81.3%)	0.973
Pre-/Peri-	141 (18.7%)	11 (19.6%)	152 (18.7%)	
Tumor Stage				
T1	464 (64.8%)	30 (49.2%)	494 (63.6%)	0.211
T2	232 (32.4%)	28 (45.9%)	260 (33.5%)	
T3	17 (2.4%)	2 (3.3%)	19 (2.4%)	
T4	3 (0.4%)	1 (1.6%)	4 (0.5%)	
Lymph Node Status				
LN-	546 (79.6%)	52 (88.1%)	598 (80.3%)	0.283
LN+	140 (20.4%)	7 (11.9%)	147 (19.7%)	
Grade				
G1	148 (19.0%)	0 (0%)	148 (17.7%)	<0.001
G2	484 (62.2%)	11 (18.3%)	495 (59.1%)	
G3	146 (18.8%)	49 (81.7%)	195 (23.3%)	
ER Staining (%)				
Low (<10%)	4 (0.5%)	7 (13.2%)	11 (1.3%)	<0.001
Medium (10-50%)	18 (2.3%)	24 (45.3%)	42 (5.0%)	
High (>50%)	768 (97.2%)	22 (41.5%)	790 (93.7%)	
Treatment Type				
Neoadjuvant therapy	96 (11.9%)	17 (27.4%)	113 (13.0%)	0.0055
Adjuvant therapy	710 (88.1%)	45 (72.6%)	755 (87.0%)	
Systemic Treatment				
ET only	184 (22.8%)	1 (1.6%)	185 (21.4%)	<0.001
ET+CT	612 (75.9%)	56 (90.3%)	668 (77.2%)	
ET,CT + TT	9 (1.1%)	5 (8.1%)	11 (1.3%)	
Chemotherapy				
TC	366 (65.2%)	18 (36.0%)	384 (62.8%)	<0.001
AC-T	134 (23.9%)	24 (48.0%)	158 (25.9%)	
Platinum containing	10 (1.8%)	1 (2.0%)	11 (1.8%)	
Other	51 (9.1%)	7 (14.0%)	58 (9.5%)	

Data presented as n (%); Unknown values excluded; N, sample size; AAPI, Asian American and Pacific Islander; AIAN, American Indian or Alaska Native; ER, estrogen receptor; ET, endocrine therapy; CT, chemotherapy; TT, targeted therapy; TC, taxane; AC-T, anthracyclines with taxane

Figure 2. Choice of treatment according to Blueprint results

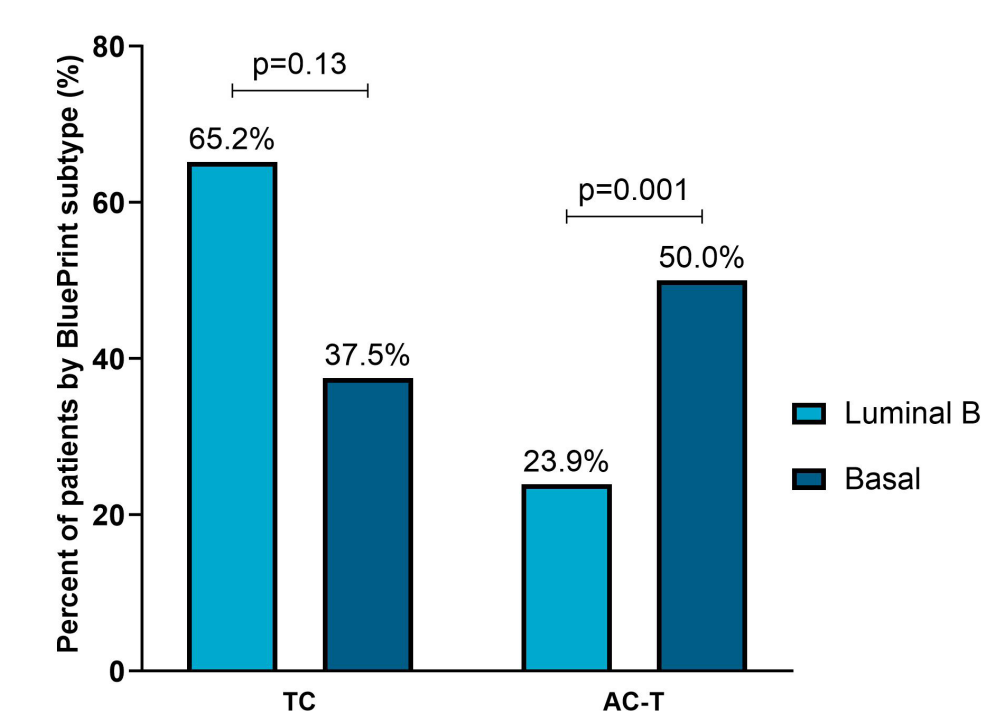

Copies of this poster obtained through the QR code are for personal use only and may not be reproduced without written permission of the authors

Conclusions

- Distinct pCR rates among breast cancer molecular subtypes underscore the utility of Blueprint in guiding treatment decisions
- Patients with Basal tumors were significantly more likely to achieve a pCR (38.2%) vs Luminal B (9.3%; p<0.001)
- Despite all patients with MammaPrint High Risk, HR+ HER2- tumors qualifying for CT, patients with HR+HER2- Basal tumors were significantly more likely to receive:
 - Neoadjuvant therapy
 - Chemotherapy
 - AC-T instead of TC
- These findings suggest that physicians used Blueprint results to guide treatment decisions